



Yes, you can.®

➤ **Invacare® *Softform® Premier Active***

Static Dynamic pressure reducing mattress

The unique patented design of the *Invacare® Softform® Premier Active* features an alternating air insert beneath the castellated foam of the tried and trusted ***Softform® Premier mattress***. The mattress retains its properties as a 'High/Very High Risk' (Waterlow score - 20)* static surface, but should a patient require stepping up to a dynamic surface, a discreet pump can be fitted to the air insert, transforming the mattress into an alternating surface delivering additional levels of pressure relief. There is no requirement to transfer the client, thereby minimising any discomfort.



Warranty

Foam: 8 Years
Cover: 4 Years



➤ **Innovative Step Up/Step Down approach**

The *Invacare® Softform® Premier Active* offers a unique and innovative 'Step Up/Step Down' approach, which avoids the need to replace a complete mattress system, therefore ensuring minimal disruption to the client. The insert consists of air cells that alternate on a ten minute cycle beneath the foam insert, delivering a very low impact at the point of change, thereby delivering high levels of comfort in addition to improved pressure reduction. The *Softform® Premier Active* is particularly suitable for those who may find a more traditional dynamic system unpleasant, due to movement and vibration.



➤ **Pump automatically assesses weight**

Sensory software within the pump can gauge the client's weight enabling pressure to be maintained at an appropriate level. In addition, the top layer of the 'High/Very High Risk' castellated foam ensures the client is fully supported in the event of a power failure. The pump is also fully alarmed in case of power failure.



How the *Invacare® Softform® Premier Active* works:



The *Invacare® Softform® Premier* is a leading pressure-reducing static mattress. The addition of the innovative air insert turns the mattress into a combined static - dynamic system.



To transform the mattress into a dynamic system and step up the client, retract the pipes from the mattress.



Fit pipes to the Active pump.



Simply switch on, and the pump automatically adjusts to the client's weight.



The client is now on an alternating surface.

Options and Features



Air insert within Softform Premier mattress foam construction

Allows stepping up and down between static and dynamic alternating systems with minimum disruption.



Sensory software within the pump can gauge weight

Pressure can be maintained at a level appropriate to the individual. Fully alarmed in case of power failure.




Inflation of air cells alternates on a 10 minute cycle

Delivers an extremely comfortable mattress surface, while preventing pressure on any one area.



High risk static mattress

In the event of a power failure, the client is still resting on a high risk pressure reducing surface, minimising the potential of pressure damage occurring.

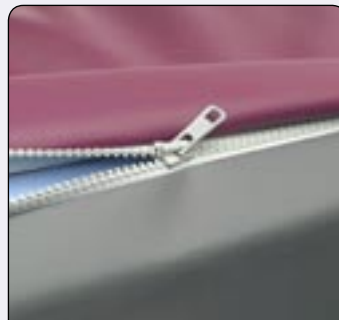

Protection against fluid ingress

The two-way stretch, waterproof and vapour permeable cover includes a full length flap covering the zip and a toughened PU coated base.

Warranty


Foam: 8 Years
Cover: 4 Years
Pump: 2 Years
Insert: 2 Years

Technical data


Colour




Fuchsia (PU)

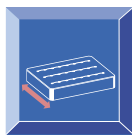

Std width:
80, 83, 85.5, 88, 90 cm

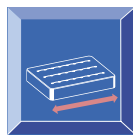

Std length:
197 cm

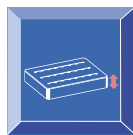


Height:
14, 15.2 cm



Weight:
13 kg



Max. Client
Weight: 247.6 kg
(39 stone)

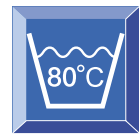

Cleaning: 80°C
Refer to instructions
for use and relevant
infection control
guidelines.

Additional sizes available on request

Manufactured to European Standards: BS EN 597-1 Cig, BS EN 597-2 Match, BS7177 :1996 Crib Source 5. Tested to pass ISO 1419:1995.

* Essential nursing care is pivotal in pressure ulcer prevention. This mattress will positively contribute to the outcome of a pressure ulcer prevention care plan. Education, clinical judgement and action based planning based on vulnerability are fundamental factors in the prevention of pressure ulcers. The Waterlow score is a formal method of assessing risk from pressure ulcer development, and should be used in conjunction with an informal assessment (informed nursing judgement). Informal assessment is considered to be of greater importance and clinical value. All information quoted is believed to be correct at time of print. Invacare® reserves the right to alter product specifications without prior consultation.



INVACARE® Limited
 Pencoed Technology Park · Pencoed · Bridgend CF35 5HZ
 Tel: +44 1 656 776 222 · Fax: +44 1656 776 220
 E-mail: uk@invacare.com · www.invacare.co.uk
 Sales Order E-mail: ordersuk@invacare.com

INVACARE® Ireland Ltd
 Unit 5 · Seatown Business Campus
 Seatown Road · SWORDS · County Dublin
 Tel. +353 1 8107084 · Fax +353 1 8107085
 Email: Ireland@invacare.com · www.invacare.ie