

Add anti entrapment sleeve if required



Use upper horizontal bar to attach remote control for easy access

Your Grab Rail/Bed Lever is now ready to use.

Occasional re-tightening of the adjustment knob maybe required.

For advice or fitting enquiries please call the number below.

Our full range of other accessories are available on our website or call and ask for a brochure.

Tel: 01869 255400 www.laybrook.com




Grab Rail/Bed Lever Assembly Instructions


The standard bed lever / grab ring will fit all bed surrounds with a depth of between 8 and 15 inches




Parts list

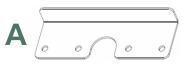

Hook (Sunken Mattress)

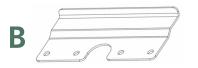

Shimmey (Raised Mattress)

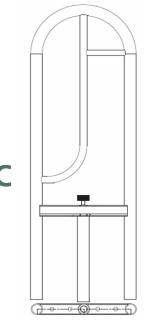

Grab Rail



Bottom Bracket (Universal)

8 x Bolts 1 x Allen Key

Attaching the Grab ring and which fitting to use...

Attach the Universal bottom Bracket (D) first using 4 screws provided



Inspect bed to see if mattress is

A -Mattress is sunken below the surround

B - Level or above surround



Sunken below the surround

For beds where the mechanism is lower than the top of the surround (mattress is sunk inside the surround) use the A hook fitting (this will clamp over the surround.




Level with the surround

B

For beds where the mechanism is level or up to 5mm lower than the top of the surround, use the B Shimmey fitting.



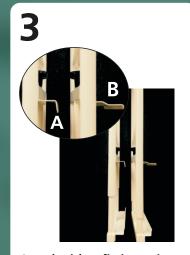

Attach either fitting using the other 4 screws provided. As shown above.





Slacken black adjustment knob anticlockwise (in centre) and attach the bottom bracket under the bed frame, hold frame upright and retighten the adjustment knob clockwise one turn, to hold in place





Ensure Tubular Grab rail is flush against the side of the bed surround (as above left) and turn black adjustment knob clockwise until tight.